

AMPUTEE INSTRUCTIONS



01 OPEN

Remove your wrap from its original packaging, and fully unfold it.



02 PLACE

Position the cup of the wrap over the amputation site, with the blue side facing in (towards your skin). Rotate the wrap as needed so that the hose connector is in a convenient location.



03 SECURE

Brace the wrap against the seat of a chair or another surface to hold it in place. One of the two straps has exposed hook fastener on one end — beginning with this strap, work your way around the wrap, securing each section to the loop side of the strap. Repeat in the other direction with the second strap, then secure to the exposed hook at the end of the first strap.



04 COMFORT

Once secured, the wrap should hold itself in place without sliding down but may not feel uniformly snug. Using the compression settings on your machine will help ensure proper contact during your session.

05 REUSE

After your first use, you can remove the wrap by loosening the second strap just enough to allow the wrap to slip off. We recommend leaving the first strap in place and only loosening the second strap to make putting your wrap on faster and easier next time.

PRODUCT AND WARRANTY INFORMATION

<https://www.nicerecovery.com/instructions-for-use>

**THIS
IS NICE**

For more information:
Call a customer representative
at (888) 815-9907 or email
info@nicerecovery.com

Location:
Boulder, CO 80301



Visit:
nicerecovery.com



Outside the United States: Please contact your local distributor.



6-1019-01 REV. A

QUICK START GUIDE

AMPUTEE WRAP

Consult a healthcare professional before using this product




- Connect the power cord to the external power supply.
- Plug the circular connector into the back of the Cooling Unit. You will hear a click when properly connected.
- Plug the power cord into the wall outlet.



- Connect the hose to the control unit. You will hear a click when properly connected.



- Place the wrap on the appropriate body part. Secure with hook fastener strips.
- Connect the hose to the wrap. You will hear a click when properly connected.

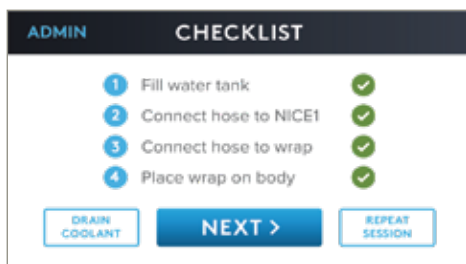

STEP 1

Turn on the power switch on the back of the device. The touch screen will illuminate and display the "Nice" logo. Press Start. Review the checklist on the touch screen. Confirm and press NEXT.




STEP 2

Select desired COLD LEVEL and press NEXT. (level 5 is the coldest setting).


Cold settings are as follows:

Level 1 = 58F (14.4C) Level 4 = 46F (7.7C)
Level 2 = 54F (12.4C) Level 5 = 42F (5.5C)
Level 3 = 50F (10C)

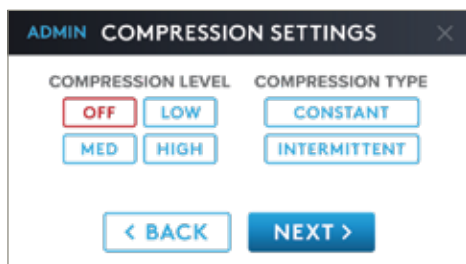

STEP 3

Select desired COMPRESSION LEVEL and COMPRESSION TYPE and press NEXT.

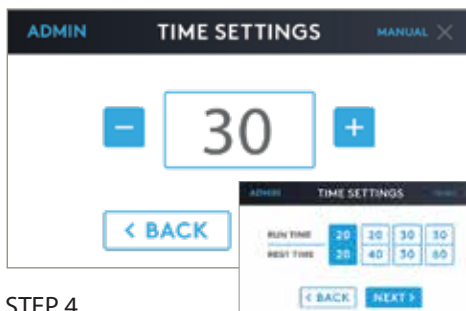


STEP 4

Choose a MANUAL time setting of 5 – 40 minutes for a single therapy session OR choose a PRESET PROGRAM for multiple sessions and press NEXT. Note: Preset timing programs are on a 10-cycle memory.



STEP 5

Review and confirm your settings by pressing START or press BACK to change the settings.

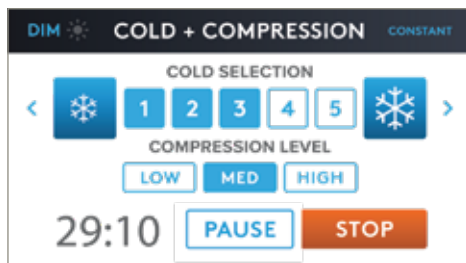


STEP 6

During your therapy session you can change the cold or compression settings simply by pressing 1 – 5 or HIGH, MED or LOW. To Dim, Press Dim switch, to brighten — touch screen.



Caution: Consult Accompanying Documents.

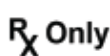

Caution: Federal law restricts this device to sale by or on the order of a physician.



Medical Device



Nice Recovery Systems LLC
2205 Central Ave, Suite A
Boulder, CO 80301



Date of manufacture



For more information call a customer service representative at (888)815-9907 or email info@nicerecovery.com



Non-sterile



Do not use if package is damaged and consult instructions for use



Does not contain natural rubber latex



Single patient multiple use



EC REP

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany